


Must be sterilized prior to use.

Face Shield Assembly Instructions

3DVerkstan 3D printed face shield head band - approved by the NIH

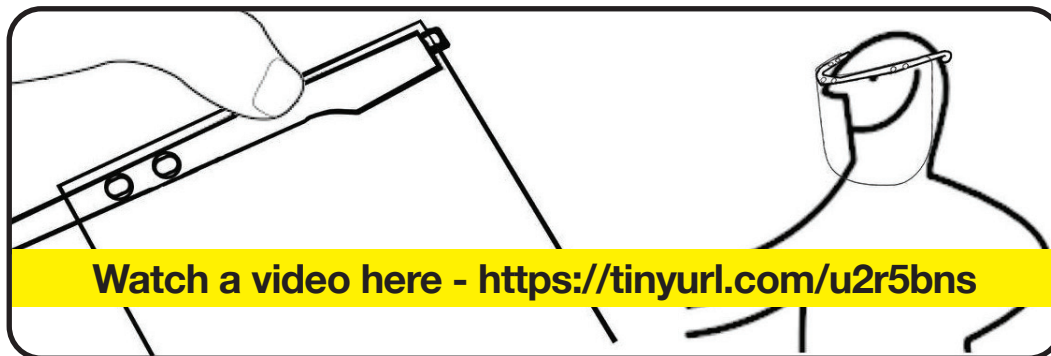

Start with the center 2 holes first and then work on the sides.

**Place your thumb just to the side of the hole
and push down with your other thumb.**

The plastic is not super strong so please be careful. The 3D printed frames are made of PLA plastic. PLA has a low melting temperature so it should not go in an autoclave. The Face Shields should not be left in a hot car.

Made by volunteers in collaboration with

St. Pete Makers • Tangible Labs • Free Dimensions • Print the Curve Flat

If you have questions please reach out to

Mo Eppley at Tangible Labs - mo@tangiblelabs.com

To make a donation or request more items visit

stpetemakers.org/covid19 or Tangiblelabs.com/covid19

Thank you for all you do! #ShieldsForHeroes